## Beta Lactamase Test

Rapid test in strip for the detection of beta-lactamase.

### INTENDED PURPOSE

Rapid test for the detection of beta-lactamase of both Gram positive and Gram negative microorganisms. This test is intended as an aid in the diagnosis, requiring further tests to complete the diagnostic results.

### DESCRIPTION

Beta Lactamase Test is a rapid test in strip for detecting the enzyme beta-lactamase which confers resistance to penicillins and cephalosporins in a wide variety of both Gram positive and Gram-negative microorganisms. *Staphylococci* are the gram-positive bacteria that most commonly produce beta-lactamase; approximately 90% or more of clinical isolates are resistant to penicillin because of enzyme production.

Gram-negative bacteria, including Enterobacteriaceae, *P.aeruginosa*, and *Acinetobacter* spp., produce dozens of different  $\beta$ -lactamase types that mediate resistance to one or more of the  $\beta$ -lactam antibiotics

### KIT CONTENT

- 30 strips of Beta Lactamase Test

### METHOD PRINCIPLE

Nitrocefin is a chromogenic cephalosporin (3-(2,4 dinitrostyl)-6R, 7R)-7-(2 thienylacetamido)-ceph-3-em-4 carboxylic acid, E-isomer) which rapidly turns from yellow to red as the amide bond in the beta-lactam ring is hydrolysed by beta-lactamases; it is sensitive to hydrolysis by all known lactamases produced by Gram positive and Gram negative microorganisms.

### MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: inoculating loop, culture media, quality control organisms.

### REAGENTS

- Nitrocefin, a chromogenic cephalosporin
- Phosphate buffer, is highly water soluble and has a high buffering capacity, but will inhibit enzymatic activity
- Dimethyl sulphoxide

### SPECIMEN

Collect specimens in sterile containers or with sterile swabs and transport to the laboratory. Process each specimen using procedures appropriate for that sample. This product is recommended for use only with pure cultures.

Refer to specific guidelines for more detailed information.

### TEST PROCEDURE

1. Take the number of Beta Lactamase Test Strips needed from the freezer and allow them to reach room temperature.
2. Touch a well isolated colony from the primary isolation medium with a sterile loop and streak it onto the portion of the strip indicated by the arrows.
3. Place the inoculated tip of the strip between the lid and the base of the inverted plate. The reaction requires moisture. It is therefore necessary to increase moisture inside the plate, or directly moisten the strip.

### INTERPRETING RESULTS

A positive Beta Lactamase test reaction is shown by the development of a pink/red colour. No colour change is observed with organisms that do not produce beta-lactamases.

To ensure correct reading the colour of the strip should be compared to an unused strip. Observe the result up to a maximum of 5 minutes and, if negative, observe again after 15 minutes. Some staphylococci can also take 1 hour before determining a colour change.

## STORAGE

At -20°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

## SHELF LIFE

1 year.

## QUALITY CONTROL

Control strain		Inoculum	Characteristic reactions
<i>Staphylococcus aureus</i>	ATCC® 29213	1 colony	Positive reaction: Appearance of a pink-red colour within 1 h
<i>Haemophilus influenzae</i>	ATCC® 35036		
<i>Neisseria gonorrhoeae</i>	ATCC® 31426		Negative reaction: No colour change
<i>Escherichia coli</i>	ATCC® 25922		

## PERFORMANCE CHARACTERISTICS

Performance testing of Beta Lactamase Test was carried out using the QC strains listed above. The results obtained met the established criteria.

## LIMITATIONS

$\beta$ -lactamase-negative, ampicillin-resistant (BLNAR) strains of *H. influenzae* have been reported.

Organisms should be tested for antimicrobial susceptibility by an acceptable, standardized method.

Certain strains of *S. aureus* produce  $\beta$ -lactamase in quantities insufficient to be detected unless induced by growth in the presence of sub-inhibitory concentrations of penicillin or semi-synthetic penicillins, such as methicillin or oxacillin.

## WARNING AND PRECAUTIONS

- 1) **For *in vitro* diagnostic use (IVD).**
- 2) **For laboratory professional use only.**
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.
- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics centre, attention has to be done to avoid erroneous data transfer.

## DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

## BIBLIOGRAPHY

See the references at the end of this document.

**TABLE OF SYMBOLS**

See the table of symbols at the end of this document.

**ORDER INFORMATION**

Product	Packaging	Ref.
Beta Lactamase Test	30 tests	88033

**Revision History**

Revision	Release Date	Change Summary
0	2023-05-15	Updated layout and content in compliance with IVDR 2017/746, version reset to revision 0

This IFU document and the SDS are available from the online Support Center:

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