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Rev.1 / 21.01.2008

Selective supplement for the preparation of DERMATOPHYTE (D.T.M.) AGAR
for dermatophytes isolation.

DESCRIPTION

DERMATOPHYTE Supplement is a freeze-dried selective supplement constituted by Chlortetracycline to use as a supplement of the culture medium DERMATOPHYTE AGAR cod. 610160 or 620160 for dermatophytes isolation.

KIT CONTENTS

Each kit contains:

- 10 vials of DERMATOPHYTE Supplement freeze-dried.
- 1 instructions sheet

PRINCIPLE OF THE METHOD

DERMATOPHYTE Supplement supports dermatophytes growth thanks to the selective activity of Chlortetracycline, antibiotic of wide spectrum which inhibit Gram-positive and Gram-negative bacteria growth.

COMPOSITION

DERMATOPHYTE Supplement		
	Content / vial	Content / l of medium
Chlortetracycline	50.0 mg	100.0 mg

PROCEDURE FOR USE

1. Aseptically reconstitute the content of one vial of DERMATOPHYTE Supplement with 5 mL of sterile distilled water .
2. Mix to complete dissolution and add to 500 mL of DERMATOPHYTE AGAR cod. 610160 or 620160, autoclaved and cooled to 45-50°C.
3. Mix well and dispense in Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical documentation for DERMATOPHYTE AGAR cod. 610160 or 620160.

QUALITY CONTROL

1. Control of the appearance: freeze-dried yellow in colour.
2. Microbiological control.

Prepare the plates using as base DERMATOPHYTE AGAR cod. 610160 or 620160 added with the content of 1 vial of DERMATOPHYTE Supplement (1 vial in 500 mL of medium). The plates are inoculated with the strains indicated in the microbiological control table. Incubation conditions: 22-30 °C for 7-14 days. Microbiological control:

	Control strains	Growth
<i>Staphylococcus aureus</i>	ATCC 25923	Inhibited
<i>Escherichia coli</i>	ATCC 25922	Inhibited
<i>Candida albicans</i>	ATCC 10231	Good

PRECAUTIONS

DERMATOPHYTE Supplement contains dangerous substances according to directives 1999/45/CE and 2001/60/CE. DERMATOPHYTE Supplement is a selective supplement to be used only for *in vitro* diagnostic use. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE

Store DERMATOPHYTE Supplement at 2-8 °C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration. Once reconstituted , the product can be stored for a maximum duration of 30 days at -20°C, shielded from light.

REFERENCES

- Allen, A.M., R.A. Drewry, and R.E. Weaver. (1970). Arch. Derm. 102: 68-70.
- Taplin, D., N. Zaias, G. Rebell, and H. Blank. (1969). Arch. Derm. 99: 1969.

PRESENTATION

Product	REF	Σ
DERMATOPHYTE Supplement	81025	10 vials

One vial is sufficient to prepare 500 mL of medium

TABLE OF SYMBOLS

 In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	 Batch code

LIOFILCHEM Bacteriology Products

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745 Fax +390858930330 Website: www.liofilchem.net

Rev.1 / 21.01.2008

E-Mail: liofilchem@liofilchem.net